Panda P3 Quick Start

Revision 1 (August 2023)



1)anda¥ Scanner

Contents

1. Connection	1
2. Computer Configuration	2
3. Installation	3
4. Account	4
5. Activation	5
6. Login	6
7. Calibration	7
8. Maintenance	8
9. Disinfection	8
10. Support ·····	9

1. Connection

We recommend to connect your scanner to your laptop in the same way as the picture below.



2. Computer Configuration

Recommended computer configuration.

CPU	Intel i7-12700H/Intel i7-11800H or above
RAM	16G/32G
Hard disk	SSD 512G or above
GPU	RTX2060/RTX3060 or above
Operating system	Windows10/11 64bit

3. Installation

Insert the USB drive in the device box to install the software. Please restart your computer after installation is complete.

apps	2023/7/31 16:45
deps	2023/7/31 16:32
en 📃	2023/7/31 15:34
📒 pt	2023/7/31 15:34
Resources	2023/7/31 15:34
🔁 zh	2023/7/31 15:34
🔊 autorun.inf	2023/5/27 19:47
🖧 favicon.ico	2023/5/27 19:47
🔏 Installer.exe	2023/7/18 21:37
Installer.exe.config	2023/7/31 16:33
🗋 manifest.json	2023/8/1 10:22
Newtonsoft.Json.dll	2021/3/18 4:03
🗟 Npgsql.dll	2018/2/28 4:36
SharpCompress.dll	2022/7/29 17:47
System.Buffers.dll	2020/2/19 18:05
System.Memory.dll	2020/2/19 18:05
System.Numerics.Vectors.dll	2018/5/15 21:29
System.Runtime.CompilerServices.U	2021/10/23 7:40
System.Text.Encoding.CodePages.dll	2021/10/23 7:45

4. Account

After completing the installation, you need a cloud platform account to store case data, share case information, and manage your device. You can log in https://en.freqtek.com to register.

🛃 FREQTY CLOUD	The 1st Choice for Digital Dentistry The latest intraoral scanning technology, start your digital journey right now!	Freqty Cloud
	CopyrightG2021 FRECITY .All rights reserved.	

Click "SIGN UP".

Si	ign up an organization account
😭 Cilnic	Lab / Dealer
*Name	Please water fail turns
4Phone	■ • +1 Please entre your noble ourber
*Password	Please other fire parentsed
*Confirm	Please entir the paraward again
*Email	Pleine enter ental
*Code	Pinnee enter the verification code
🗆 Read a	ind agree our (Privacy Statement and Terms of Service)
	Ratum Skipi Lip

Choose "Clinic" or "Lab" according to your actual situation, and then fill in the columns below carefully.

5. Activation

After completing the registration, use the email you filled in during registration and the password you set to access the https://en.freqtek.com Complete login. When you login your account for the first time it will pop up a guide to help you bind your device.

FREQTY	Case Statistics Downloaded	Status	Case Type	Labs & De	evices	Partner Labs	Total: 0
	Total 0 😹 Day 0 Week 0 Month 0 Y	1 Register Device	Orthodontics Restoration O Set Data Receiver	0 0 0 Start Scanning	egistered Device		
	Uploaded Cases Statistics Countries	* Device S/N Imput * License Code Imput Notes Imput	the Device Number be Device Number the device code be device code of ender code signal the deviception about the device		<u>- me - 1</u>		
© PARTNERS © ACCOUNT © TRASH	ш	Whether to Encrypt	D Bind				
	as Skip	No Prompts)			Next		
Rew Device							
	0 - 0 - 0 - 0 - 0 - 0 - 0 - 0 - 0 - 0 -	0 2013-04-08	0 00 00 00 00 00 00 00 00 00 00 00 00 0	o 2021-00-11	2003-00-12		

6. Login

After completing the registration and device binding, please return to the desktop, double click "PandaCenter.exe" (c) to open the software.

🖧 FREQTY CLOUD	× Welcome to login
High-speed upload, easy to use, One-click upload	User Password
Email notifications Efficient and convenient	Username/Email
	Password Forget Password
***	Login Sign up
	English(United States)
	freqty.com 1.1.1(en)

7. Calibration

The new device requires calibration before it can be used normally after first activation, with the aim of ensuring that the device still has good accuracy after long-distance transportation. Next part is the specific process of calibration.

NTER	OR	DER	MANAG	SEMENT					
ORDER	Patient)	lanse	P	G O Send O Receive	C Recycler			Z_ PROVINCE	L+ CHENTE ONDER
EVICE	111	Inder Type	v					Re	fresh List Download All
		NUMBER	PATIENT	DOCTOR	SENDER	TYPE	CREATE TIME 0	STATUS	OPERATE
a TWARE						No data			
ary) Touron									

Click (Usually, there is a red dot prompt for the initial device binding).



Click "CALIBRATION" to start calibration.

8. Maintenance

It is prohibited to use a three ways syringe to blow inside the tip to prevent dust from being blown into the photoelectric system inside the device. If there are stains on the lens glass, please dip a small amount of anhydrous alcohol and shake it dry slightly before gently wiping the glass. During the wiping process, please pay attention to avoid damaging the glass and affecting the imaging.

9. Disinfection

Device body: Use alcohol with a concentration of 75% or more or other surface disinfection products to wipe and disinfect. Be careful not to soak in any liquid, otherwise it may cause damage. Tip: The tip is designed to be reusable, and for hospital infection prevention and control, please be sure to disinfect and use it according to regulations.

Put it into a high-temperature and high-pressure sterilization pot, and the recommended disinfection temperature is 134 $^{\circ}$ C for at least 6 minutes (121 $^{\circ}$ C for at least 15 minutes).

10. Support

If there are any problems with the device that you cannot solve, please do not disassemble the device without permission.

If you have any questions or needs, please contact us through the following methods.

Support@freqty.com

(+86) 400-999-6806 (Global)



⊕ www.panda-scanner.com
∞ info@panda-scanner.com
Chengdu Ziyang | Suzhou | Ningbo | Brazil | USA